

# ConvaTec™

Hydrocolloid Sealing Strips

Supporting the optimal  
seal for NPWT



# Establishing an effective seal is crucial to ensure effective delivery of Negative Pressure Wound Therapy.

Application of negative pressure to difficult to dress anatomical locations along with patient factors such as perspiration, mobilisation and showering can lead to challenges in maintaining a long-lasting air-tight seal. ConvaTec™ Hydrocolloid Sealing Strips are thin, flexible and are designed to assist in creating the optimal seal to ensure your patients receive effective NPWT.



## Product Description:

Sterile thin sealing strips consisting of a layer of hydrocolloid contained within an adhesive polymer matrix and an outer layer of polyurethane film.

## Indications:

Use on intact skin as a securement for patients receiving NPWT. The sealing strips can be used to help seal leaks around the periphery of negative pressure dressings.

## Benefits:

- Absorptive hydrocolloid properties help to maintain NPWT seal in the presence of exudate or sweat and help to reduce the risk of peri-wound maceration.
- Thin polyurethane film provides a bacterial, viral and showerproof barrier.
- Easy to use, mould and can be cut to shape for awkward to dress areas.
- Sealing strips can be removed without damaging surrounding skin.
- 7-day wear time.


## Application




1. Remove the backing paper and film, minimising finger contact with the adhesive surface.



2. Apply the sealing strip with a 1cm overlap on to the periphery of the negative pressure dressing. Avoid creasing and overlapping of the sealing strip and do not stretch or apply under tension.

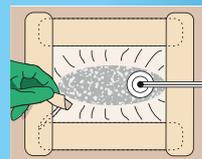

3. If large wrinkles are visible within the negative pressure dressing, place the sealing strip so it runs in-line along the length of the wrinkle and not across the wrinkle. Take care not to trap the negative pressure tubing.

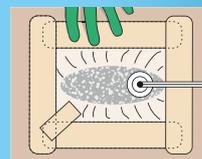

4. Mould into place with hand for a secure adhesion.

**Note:** Maximum recommended wear time is up to 7 days. If a seal cannot be achieved consult the Instructions for Use for the NPWT device that you are using.

## Ordering Information:

Product Code	Size	Pack Size
422308	5 x 20cm	10

To find out more about the ConvaTec portfolio or to arrange a visit from your local ConvaTec representative: Call: 0800 289 738(UK) or 1800 946 938 (ROI)

[www.convatec.co.uk](http://www.convatec.co.uk)

®/™ ConvaTec Hydrocolloid Sealing Strips are trademarks of ConvaTec Inc. All other trademarks are the property of their respective owners. 2019 ConvaTec Inc. AP-O20874-MM



Negative Pressure Wound Therapy